

Partial Sample Version 1

If an athlete cannot provide the required amount of urine, the partial sample needs to be temporarily and securely sealed (in accordance with the International Standard of Testing, WADA ISTI).

Contents

- 1 numbered security plastic bag with 3 receipts
- 1 blue plastic stopper
- 1 handling information sheet



Handling information

- Fill the collected urine into the B-bottle and close the partially filled bottle with the blue plastic cap.

Remove the red ring from the bottle neck.
The A-bottle remains unopened.



- Place the bottles back into the styrofoam box / cardboard box and then put all components, including the numbered security cap, into the security plastic bag.



- Seal the security plastic bag by removing the backing paper from the tamper-evident void tape and press it firmly together.

(Each numbered security bag has 3 receipts for e.g. the athlete or the DCO).

- The BEREG-KIT is now temporarily securely sealed.



Attention:

Storage conditions: at moderate temperatures between 0 ° C and + 40 ° C

The manufacturer shall not be held liable if the Partial sample Version 1 was used beyond the manufacturers given limitations and handling instructions.

Please note that any instructions for use published prior to this dated guide are no longer valid!