

Handling instruction for the urine collection vessel

Before using the collection vessel

- Open only the lower part of the packaging which is marked with “①”.

The upper part containing the lid remains closed.

- Tear open on the perforation.

- Take out the vessel.

Do not touch the inside of the vessel.



Filling of the collection vessel

- Fill the collection vessel with urine according to your guidelines.



Closing of the collection vessel

- Place the vessel on a firm and even surface.
- Open the upper part of the packaging which is marked with “②” and take out the green lid.
- Place the green lid onto the collection vessel.
Do not remove the cover from the spout yet!
- Now press with the palm of your hand onto the lid of the collection vessel until it is firmly closed.
- A double click ensures proper and leak-proof closure of the vessel.



Decanting the urine

- Before pouring the collected urine into the glass bottles, remove the spout cover from the green lid.
- **IMPORTANT:**
To decant the collected urine easily into the bottle and to avoid spillage, make sure that the spout of the collection vessel is resting on the rim of the glass bottle.
- Please keep bottle and vessel at an angle (see picture).



The manufacturer shall not be held liable if the collection vessel was used beyond the manufacturers given limitations and handling instructions. Please note that any instructions for use published prior to this dated guide are no longer valid! The latest version of the handling instruction can be found on our website: www.berlinger.com